

Chamber Cleaning Procedure

SCIL VET ABC PLUS

This procedure should be done once a month



1. Log in as VET and press the Maintenance Button
2. Select the Service sub menu at the top left of the screen

3. Press the Customer Service Button



4. Press the purple button beside Chambers Draining

5. Press the Start Cycle Button



6. Open up the Front Door of the Vet ABC Plus

7. Use the Vet ABC 2.5mm Allen Key screw driver to loosen the one screw holding the black cover on. Once the screw is loosened about 3 turns the cover will lift up. Pull the cover towards you to remove it.



8. Use a dampened KimWipe with distilled water to clean around the rim of each chamber. Be VERY CAREFUL not to allow any of the crystallized flakes to land inside the chambers



9. Put 3mL of distilled water in each of the chambers and start the Chambers Draining cycle again
10. Slide the black cover back on making sure the cover sits flush against the frame and no tubes are pinched. Run a concentrated cleaning once you have finished.