

Extensive Cleaning Procedure Vet abc

- 1. Go to # 4 – Service on your menu, select “Drain Chambers”**
- 2. Open cover door**
- 3. Pour 5 ml of 100% bleach into mixing chamber**
- 4. Go to # 4 – Service, select “Transfer”**
- 5. Pour 5 ml of 100% bleach into RBC (red blood cell) chamber**
- 6. Run 10 BACKFLUSH (# 4 Service - # 1) cycles**
- 7. Let unit sit for 3 – 4 hours or overnight**
- 8. Run 10 BACKFLUSH (# 4 Service - # 1) cycles**
- 9. Run one Concentrated Cleaning procedure**
- 10. Run one Prime Cycle (# 4 Service - # 3 Reagent Pack – select Prime)**
- 11. Run a start up cycle**
- 12. If start up passes then everything is fine**
- 13. If start up failed, please call our service line.**

Toll Free Support Line at 1-866-382-6937

OR Larry Cochrane at 705-796-4888

*Vet Novations Canada Inc., 131 Saunders Road, Unit # 5, Barrie, ON, L4N 9A7
Tel.: (705) 726-9995 Fax: (705) 726-9202*

