

HOW TO REMOVE A MIXING CHAMBER BLOCKAGE

1. Open the front door of the vet abc
2. If there is a black cover over your mixing and RBC chamber you must remove it by loosening the top and bottom screws and sliding the cover upwards, then pulling it off.




Screw # 2 is located inside the black box. You will need to put the vet abc on the edge of a table and bend down to see it.

3. Pull the mixing chamber out of the clip.



4. Remove the tubing from the T.

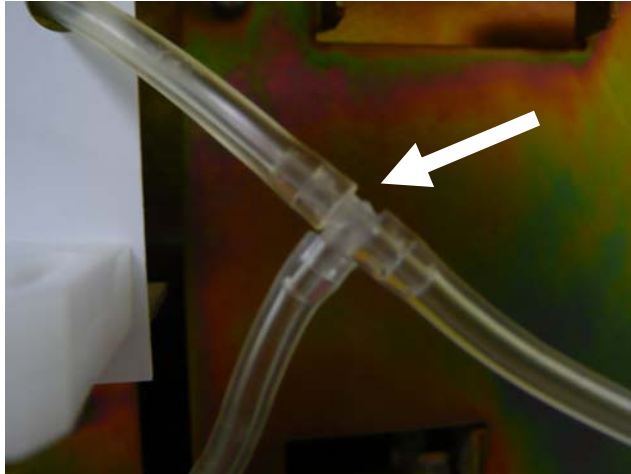

5. Inspect the T for clots.
6. If you see a clot, suck it out using a syringe.
7. Flush out the T out using a syringe with bleach in it.
8. Reattach the tubing as it was originally.
9. Push the mixing chamber back into the clip.
10. Put the black cover back on. Be sure not to pinch any tubing.
11. Run one concentrated cleaning
12. Run a Start Up Cycle
13. Run a blood control. If your controls are not in range fax your blood control results to Vet Novations at 705-726-9202.