

## ICONS

SCIL Vet ABC Plus+

### MENU




**Sample ID** – Enter data for running a sample



**Quality Control** – (Quality control data, calibration)



**Results Archive** – Stored and searchable results up to 1000 results

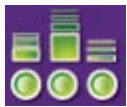

**Maintenance** – (Reagents, archive, stats, logs, service)



**Status** – Displays all alarms, run a Start Up from this screen



**Exit** – Change user, restart or shut down machine



**Help** – Provides help with the current screen

# Vet Novations Technical Support



## ACTION



**Start up** – Initiates a Start Up Cycle



**Shutdown** – Shuts machine down



**Restart** – Restarts machine



**Logout** – Disconnects current user and displays the Log In screen



**Edit** – Allows user to begin making changes within the current screen



**OK** – Validates an action



**Cancel** – Cancels an action



**Reset** – Resets the entered fields or bypasses the need to archive



**Delete** – Deletes the selected item

# Vet Novations Technical Support



**Back** – Returns to previous displayed screen




**View** – Displays additional information



**Print** – Forces the machine to print the current screen



**Send** – Sends data to computer

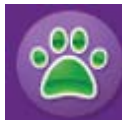

**Species** – Allows user to change species



**OK** – Confirms an action



**Cancel** – Cancels an action



**Emergency Stop** – Stops a cycle in progress



**New User Button** – Used to create a new user in the settings menu

# Vet Novations Technical Support



**Keyboard** – Opens virtual Keyboard



**Valid** – Validates keyboard entry and moves to next field



**Close** – Closes keyboard after entries are all complete

## QUALITY CONTROL



**Import Button** – Used to import Quality Control Target values



**Control History** – Shows the control history stored in machine



**QC Expired** – Loaded QC data is out of date



**Inter QC time has elapsed** – Instrument expects a QC to be run



**Invalid QC** – Some of the parameters are out of range on the last ran QC



**QC Passed** – All parameters are within range

# Vet Novations Technical Support



**QC Icon** – displayed when proper QC lot # is entered in Sample ID



**Calibration Creation**- Used to calibrate machine, opens menu

## REAGENT



**Reagent Replace** – Used to replace the current reagent pack with a new one



**Prime** – Used to prime the machine with reagent



**Start Cycle** – Starts a selected cycle such as priming etc.

## SERVICE



**Settings** – (Date, time, printer, network, species, alarms)



**Customer Service** – (Prime, back flush, concentrated clean, chamber draining)



**Service Cycles** – (Needle, carriage, valves etc.)

# Vet Novations Technical Support



## ARCHIVE



**Complete Archive** – Archives all data found in machine



**Incremental Archive** – Archives the last 30 days of information



**Restore Button** – Settings and analyzer settings can be restored to an older configuration