

Checklist for Investigating Aberrant Results on the Vet ABC

Please follow these steps if possible before you contact us – it will help us provide more expedient solutions.

1. Calibration and Controls
 - Should be run to give preliminary evaluation of machine
 - We need copies of the results
 - If you have questions – please feel free to call our office first
2. CBC results
 - Copies of results from the Vet ABC including histograms
 - Any reference lab values for that specific CBC
 - Manual smear results if done
3. Sample handling details
 - Age of sample when run on the ABC
 - Needle size for venipuncture
 - Any difficulty acquiring sample?
 - Any pathology with sample – i.e. icterus, lipemia, hemolysis?
4. When was the last time the Vet ABC was serviced?

***Please fax the information above to 705-726-9202
If you have any questions, please call 1-866-382-6937***

